

EVIDENCE

FRENCH MINISTRY OF HEALTH PROSPECTIVE MULTICENTER STUDY USING BIO ACTIVE STENTS COATED WITH TITANIUM NITRIDE OXIDE



TRIAL DESIGN

- Prospective, multicentric study: 21 centres
- 356 patients enrolled
- 100% Data Monitoring by independent CRO
- Primary Endpoint: MACE at 12 months
- No exclusion criteria

PATIENTS CHARACTERISTICS


		n=356
Age (mean ± SD)		67.4 ± 12
Male sex		77.3%
Diabetes		18.2%
Hyperlipidemia		62.6%
Smoking		38.2%
Arteriopathy		11.4%
Previous stroke		3.5%
Stable Angina		60.6%
Acute Coronary Syndrome		39.4%
STEMI		18.2%
NSTEMI		21.2%

PROCEDURAL CHARACTERISTICS

		n=356
Lesion location	CD	28%
	IVA	48%
	CX	24%
Lesion type	A	17.3%
	B	75.7%
	C	7%
Acute Procedural Success		100%
RVD (mm)		2.95 ± 0.45
Lesion length (mm)		11.8 ± 3.6
Stent length (mm)		14.4 ± 4.5
Number of stents per lesion		1.3 ± 0.5

EVIDENCE FRENCH MINISTRY OF HEALTH PROSPECTIVE MULTICENTER STUDY USING BIO ACTIVE STENTS COATED WITH TITANIUM NITRIDE OXIDE

12-MONTH RESULTS

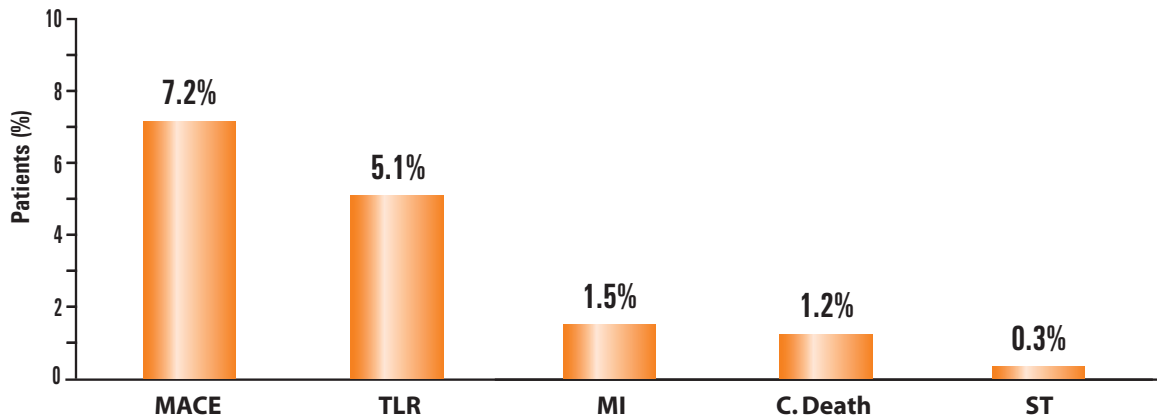

CONCLUSIONS

The EVIDENCE (M.O.H study) clinical results reaffirm:

- The low rate of MACE obtained with the TiTAN2 B.A.S* at 1 year
- The excellent Efficacy/Safety ratio provided with the TiTAN2 B.A.S

*B.A.S: Bio Active Stent

PARTICIPATING CENTERS

Principal Investigator - Pr J.-M. LABLANCHE - Hôpital Cardiologique, Lille - Dr M. ANGIOI - Hôpital de Brabois, Nancy - Dr P. BARRAGAN - Polyclinique Les Fleurs, Ollioules - Dr S. CATTAN - Centre Hospitalier, Montfermeil - Dr F. COLLET - Hôpital Privé Clairval, Marseille - Dr P. DUPOUY - Hôpital Privé, Antony - Dr P. Durand - Clinique Bizet, Paris - Dr J. FAJADET - Clinique Pasteur, Toulouse - Dr G. FINET - Hôpital Cardio Vasculaire, Lyon - Dr B. GERARDIN - Clinique A. Paré, Neuilly - Dr A. GOMMEAUX - Polyclinique Bois Bernard, Lille - Dr B. KARSENTY - Clinique St Martin, Bordeaux - Dr C. LOUBEYRE - Polyclinique St Laurent, Rennes - Dr P. MEYER - Institut A. Tzanck, Saint-Laurent-du-Var - Dr M.-C. MORICE - Institut Hospitalier J. Cartier, Massy - Dr M.-E. MOULICHON - Clinique St Pierre, Perpignan - Dr S. SAYAH - Hôpital de Compiègne, Compiègne - Dr M. SILVESTRI - Polyclinique Rambot, Aix-en-Provence - Dr X. TABONE - Hôpital J. Cœur, Bourges - Dr T. UNTERSEEH - Hôpital C. Gallien ICPS, Quincy-Sous-Senart - Dr O. WITTENBERG - Hôpital Privé Beaugregard, Marseille



HEXACATH

4, Passage Saint-Antoine
92508 Rueil-Malmaison Cedex
Tel: +33 (0)1 41 39 01 92
Fax: +33 (0)1 41 39 01 93
Web: www.hexacath.com

